



Amelogen Plus Syringe Kits*

Complete Syringe Kit REF/UP 3254

1 - each Dentin shade: A1, A2, A3, A3.5, A4, A5, B1, C2,
1 - each Enamel shade: Opaque White, Enamel White, Enamel Neutral, Enamel Gray, Translucent White, Translucent Gray, Translucent Orange, 1 - Ultradent Bonding System, 1 - 1.2ml PermaSeal syringe, 1 - Shade guide, 1 - Full size syringe organizer, 1 - Quad key, 10 - Black Mini Brush tips, 10 - Blue Micro tips, 20 - Inspiral Brush tips

Cosmetic Kit REF/UP 315

1 - each Dentin shade: A1, A2, A3, 1 - each Enamel shade: Opaque White, Enamel White, Enamel Neutral, Translucent White, 1 - Ultradent Bonding System, 1 - 1.2ml PermaSeal syringe, 1 - Shade guide, 1 - Half size syringe organizer, 1 - Quad key, 10 - Black Mini Brush tips, 10 - Blue Microtips, 20 - Inspiral Brush tips

Natural Kit REF/UP 3013

1 - each Dentin shade: A2, A3, A4, C2 1 - each Enamel shade: Enamel Neutral, Enamel Gray, Translucent Gray, 1 - Ultradent Bonding System, 1 - 1.2 ml PermaSeal syringe, 1 - Shade guide, 1 - Half size syringe organizer, 1 - Quad key, 10 - Black Mini Brush tips, 10 - Blue Micro tips, 20 - Inspiral Brush tips

Basic Kit REF/UP 3098

1 - each Dentin shade: A1, A2, A3, A4, A5, B1, C2, 1 - Ultradent Bonding System, 1 - 1.2ml PermaSeal syringe, 1 - Shade guide, 1 - Half size syringe organizer, 1 - Quad key, 10 - Black Mini Brush tips, 10 - Blue Micro tips, 20 - Inspiral Brush tips

*SHELF LIFE18 MONTHS. Refrigerated.

Amelogen Plus Syringe Refills[†] (2.5g)

Dentin Shades		Enamel Shades	
REF/UP	Shade	REF/UP	Shade
9030	A1	9037	Opaque White
9031	A2	9038	Enamel White
9032	A3	9039	Enamel Neutral
9028	A3.5	9040	Enamel Gray
9033	A4	9041	Translucent White
9034	A5	9042	Translucent Gray
9035	B1	9043	Translucent Orange
9036	C2		

[†]SHELF LIFE: 36 MONTHS. Refrigerated.

Amelogen Plus Singles Kits*

Complete Singles Kit REF/UP 4525

1 - each Dentin shade 10pks: A1, A2, A3, A3.5, A4, A5, B1, C2,
1 - each Enamel shade 10pks: Opaque White, Enamel White, Enamel Neutral, Enamel Gray, Translucent White, Translucent Gray, Translucent Orange, 1 - Ultradent Bonding System, 1 - 1.2ml syringe PermaSeal, 1 - Shadeguide, 1 - Combo pack tips, 10 - Black Mini Brush tips, 10 - Blue Micro tips, 20 - Inspiral Brush tips

Cosmetic Singles Kit REF/UP 3104

1 - each Dentin shade 10pks: A1, A2, A3, 1 - each Enamel shade 10pks: Opaque White, Enamel White, Enamel Neutral, Translucent White, 1 - Ultradent Bonding System, 1 - 1.2ml syringe PermaSeal, 1 - Shade guide, 1 - Combo pack tips, 10 - Black Mini Brush tips, 10 - Blue Micro tips, 20 - Inspiral Brush tips

Natural Singles Kit REF/UP 3239


1 - each Dentin shade 10pks: A2, A3, A4, C2, 1 - each Enamel shade 10pks: Enamel Neutral, Enamel Gray, Translucent Gray, 1 - Ultradent Bonding System, 1 - 1.2ml syringe PermaSeal, 1 - Shade guide, 1 - Combo pack tips, 10 - BlackMini Brush tips, 10 - Blue Micro tips, 20 - Inspiral Brush tips

Basic Singles Kit REF/UP 3209

1 - each Dentin shade 10pks: A1, A2, A3, A4, A5, B1, C2, 1 - Ultradent Bonding System, 1 - 1.2ml syringe PermaSeal, 1 - Shade guide, 1 - Combo pack tips, 10 - Black Mini Brush tips, 10 - Blue Micro tips, 20 - Inspiral Brush tips

Amelogen Plus Singles Refills[†] (0.3g)

Dentin Shades		Enamel Shades	
REF/UP	Shade	REF/UP	Shade
8010	A1	8021	Opaque White
8011	A2	8022	Enamel White
8013	A3	8023	Enamel Neutral
8024	A3.5	8012	Enamel Gray
8014	A4	8016	Translucent White
8015	A5	8018	Translucent Gray
8017	B1	8019	Translucent Orange
8020	C2		

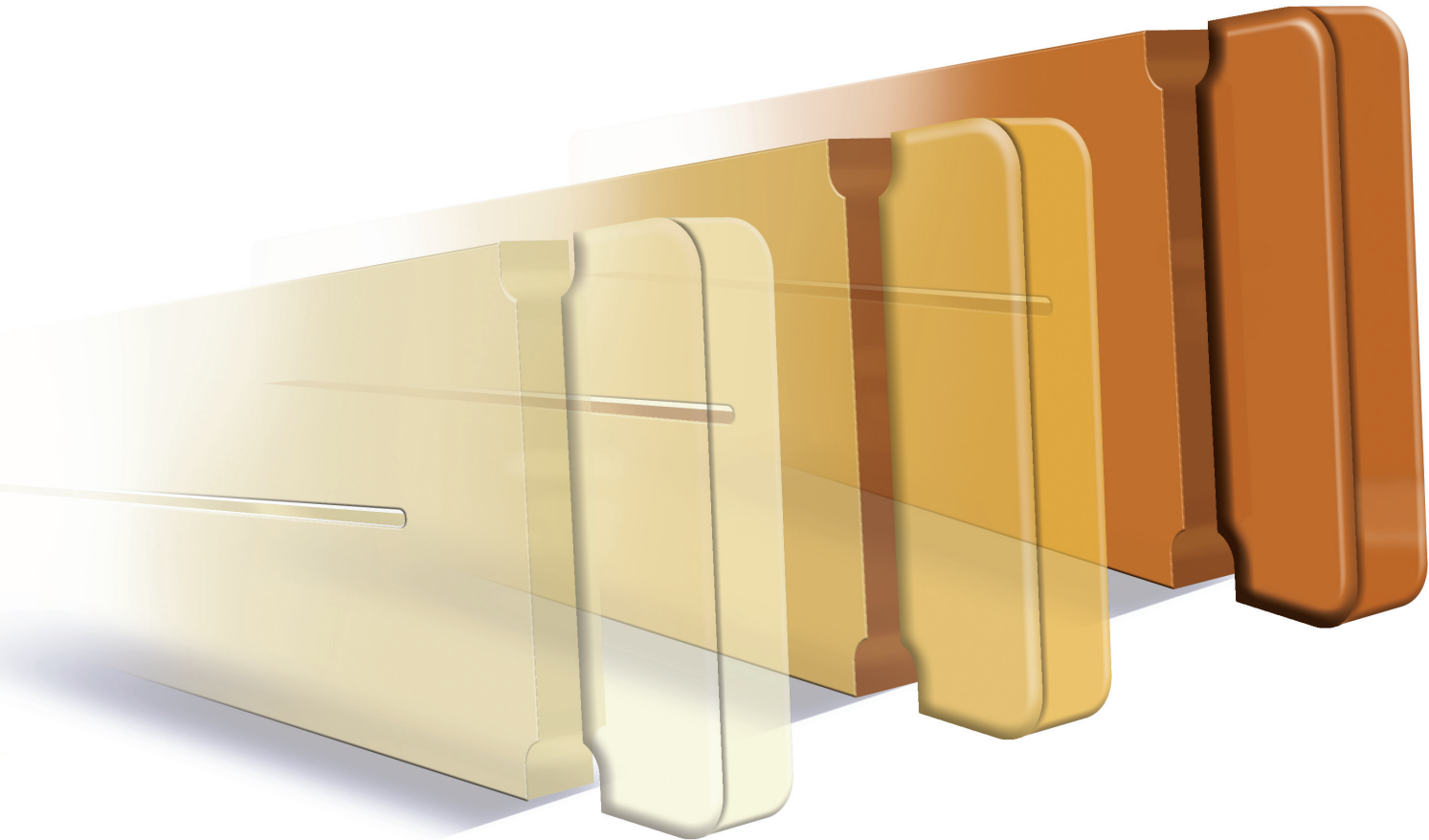

amelogen[®]plus
simple technique guide



ULTRADENT
PRODUCTS, INC.

www.ultradent.com • 800.552.5512

68308.1 020110
© 2010 Ultradent Products, Inc. All rights reserved.


amelogen[®]plus

simple technique guide*

A simple, affordable composite material offering strength and esthetics, Amelogen Plus is ideal for anterior and posterior restorations. Available in 15 shades, this highly polishable and sculptable micro-hybrid composite system allows you to easily create life-like restorations.

1 Single-Shade Restoration:

- Step 1: Select the overall tooth shade.
- Step 2: Etch and bond prepared tooth.
- Step 3: Place the first layer of composite 0.5-1mm thick. Place remaining layers in 2mm increments, replicating the anatomy of the surrounding tooth structure. Light cure 20 seconds between each layer.
- Step 4: Shape, finish, polish, and seal.

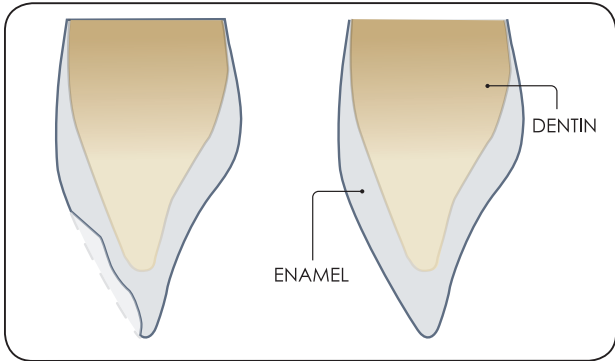


Shade and Translucency

DENTIN SHADES	ENAMEL SHADES	
A1	ENAMEL WHITE	BODY
A2	ENAMEL NEUTRAL	
A3	ENAMEL GRAY	
A3.5	TRANS WHITE	TRANSLUCENT
A4	TRANS GRAY	
A5	TRANS ORANGE	
B1	OPAQUE WHITE	OPAQUE
C2		

2 Two-Shade Restoration:

- Step 1: Select the dentin shade by matching the exposed dentin or cervical third of the tooth.
- Step 2: Select enamel shade by matching adjacent enamel.
- Step 3: Etch and bond prepared tooth.
- Step 4: Place the first 0.5-1mm layer of dentin composite. Place remaining layers in 2mm increments, replicating the anatomy of the surrounding tooth structure. Light cure 20 seconds between each layer.
- Step 5: Place enamel composite layer in 1-2mm increments, replicating the anatomy of the surrounding enamel tooth structure. Light cure 20 seconds between each layer.
- Step 6: Shape, finish, polish, and seal.

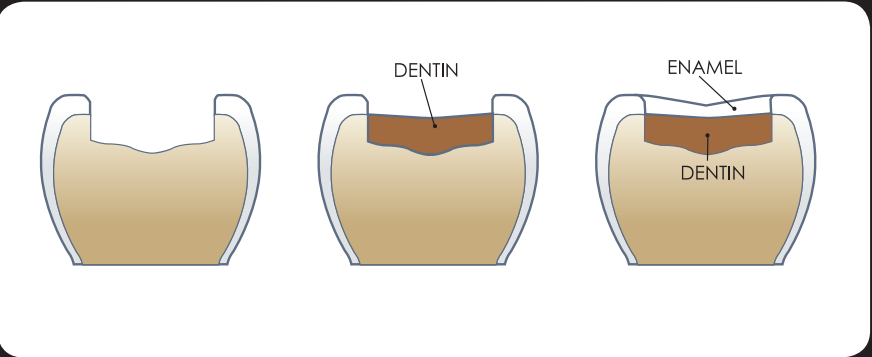

*Note: Refer to Amelogen Plus Instructions For Use for complete instructions on how to use this product.

ULTRADENT
PRODUCTS, INC.

www.ultradent.com • 800.552.5512