




ENDODONTIC POST CEMENTATION GUIDE

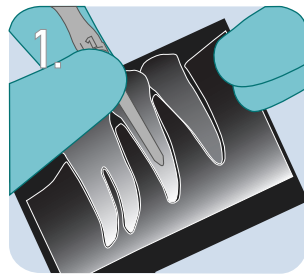


1. Determine post size and length using a try-in post or x-ray and clinical judgement.

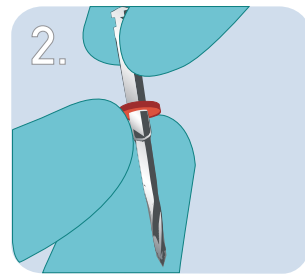


2. Place a rubber stop on UniCore® Drill at desired length.

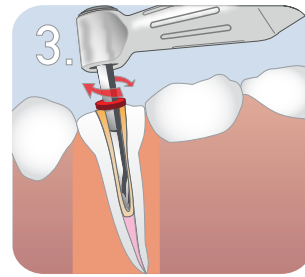


3. Position UniCore tip in the pilot hole. Using light pressure, follow the obturation material to the length indicated by rubber stop. Keeping the drill at full speed, withdraw from the canal.

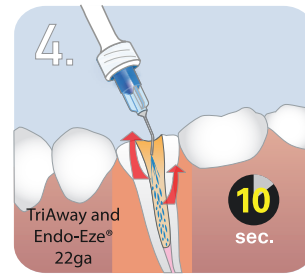


4. Use TriAway Adapter with Endo-Eze 22ga Tip to rinse debris out of post space with water. **10 sec.**

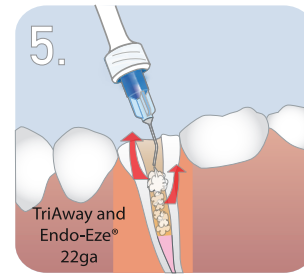


5. Blow out excess water with TriAway Adapter and Endo-Eze 22ga Tip to dry the canal.

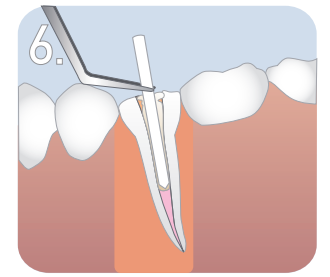


6. Verify post size by placing the corresponding UniCore Post and seat to length.

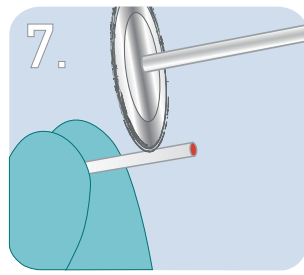


7. Trim post to appropriate length using a diamond disc.

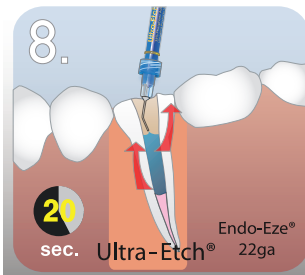


8. Etch space for 20 seconds with Ultra-Etch using the Endo-Eze 22ga Tip. **20 sec.**

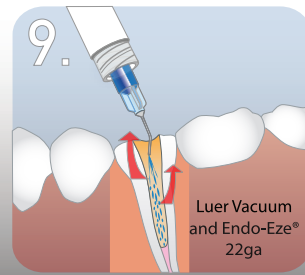


9. Suction excess etchant with Luer Vacuum Adapter and Endo-Eze 22ga Tip. **20 sec.**

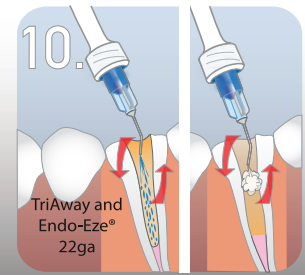


10. Use TriAway and Endo-Eze 22ga Tip to rinse thoroughly with water and lightly air dry, leaving the post space slightly damp.

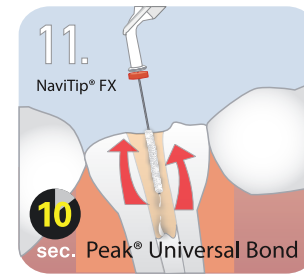


11. Use the NaviTip FX Tip to place Peak Universal Bond in the canal for 10 seconds. **10 sec.**

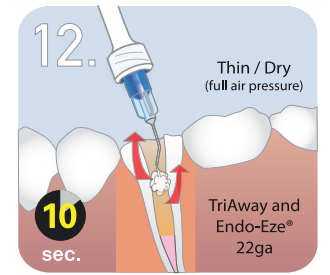


12. Thin and dry Peak Universal Bond with full air pressure for 10 seconds with the TriAway Adapter and Endo-Eze 22ga Tip. **10 sec.**

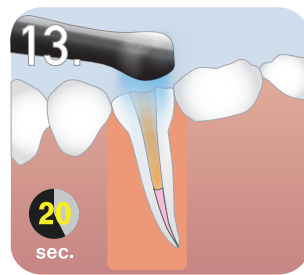


13. Light cure adhesive in post space for 20 seconds. **20 sec.**

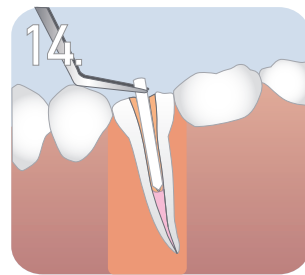


14. Using the UniCore post or a paper point, ensure post space is not obstructed with Peak Universal Bond.

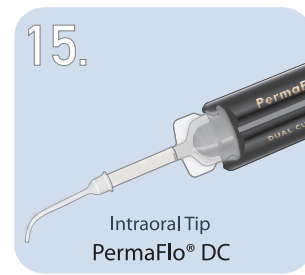


15. Attach the Intraoral Tip to the mixing tip of the PermaFlo DC syringe.

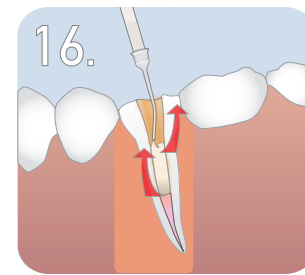


16. Use the Intraoral Tip to deliver mixed PermaFlo DC into post space beginning from the bottom of the post space.

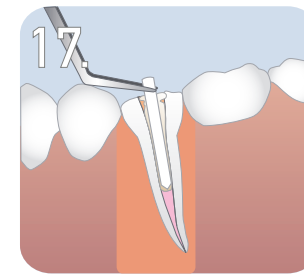

17. Insert post and seat.

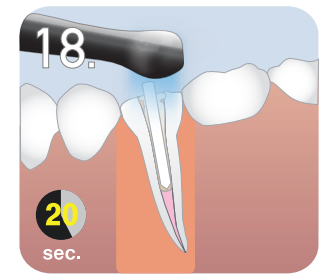


18. Light cure PermaFlo DC in canal through post for 20 seconds. **20 sec.**

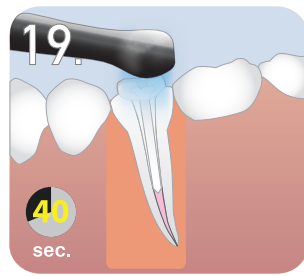


19. Flow PermaFlo DC around post for core build-up. If fill starts to slump, light cure 2-3 seconds. Repeat with each additional layer. Final cure with VALO for 20 seconds, or 40 seconds if using another curing light with less than 600 mw/cm². Proceed with crown preparation or restorative procedure. **40 sec.**

Flow PermaFlo DC around post for core build-up. If fill starts to slump, light cure 2-3 seconds. Repeat with each additional layer. Final cure with VALO for 20 seconds, or 40 seconds if using another curing light with less than 600 mw/cm². Proceed with crown preparation or restorative procedure.

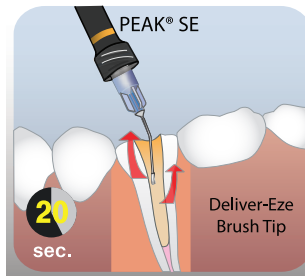

20. Apply Peak SE into the canal using the Deliver-Eze 22ga Brush Tip. Scrub for 20 seconds. **20 sec.**

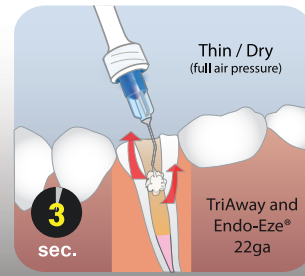

21. Thin and dry Peak SE with full air pressure for 3 seconds with the TriAway and Endo-Eze 22ga Tip. **3 sec.**

SELF-ETCH TECHNIQUE (Replaces steps 8-10)

Apply Peak SE into the canal using the Deliver-Eze 22ga Brush Tip. Scrub for 20 seconds. Thin and dry Peak SE with full air pressure for 3 seconds with the TriAway and Endo-Eze 22ga Tip.

PermaFlo DC
QUALITY SEAL. SUPERIOR HOLD. ULTRADENT CEMENTS.



PermaFlo® DC

ENDODONTIC POST CEMENTATION GUIDE



BONDED POST KIT

#1319 – PermaFlo DC Bonded Post Kit

- 2 – 5ml Dual Barrel PermaFlo DC A2 syringes
- 2 – 1.2ml Ultra-Etch
- 2 – 1.0ml Peak SE
- 2 – 1.2ml Peak Universal Bond
- 5 – Luer Vacuum Adapters
- 20 – TriAway Adapters
- 20 – Intraoral tips
- 20 – Mixing tips
- 20 – Navi FX 17mm tips
- 40 – Endo-Eze 22ga tips
- 20 – Deliver Eze-Brush tips



SHELF LIFE: 24 MONTHS. PERMAFLO DC. Refrigerated.
SHELF LIFE: 18 MONTHS. PEAK UNIVERSAL BOND. Refrigerated.

SHADE KITS

- #5912 - A2
- #5914 - Translucent
- #5913 - A3.5
- #5915 - Opaque White

- 1 – 5ml Dual Barrel syringe
- 20 – Mixing tips
- 20 – Intraoral tips



SHELF LIFE: 24 MONTHS. Refrigerated.

800.552.5512
ULTRADENT.COM

 **ULTRADENT**
PRODUCTS, INC.

© 2012, Ultradent Products Inc. U.S. and International patents pending and granted. All rights reserved.

68482.3 010512

 **ULTRADENT**
PRODUCTS, INC.