Amelogen® Plus	EN	DE	FR	NL	IT
----------------	----	----	----	----	----

Composite Restorative Material

Description and Indications for use:

Amelogen Plus is a tooth shade resin composite material (Bis-GMA based) to be used for posterior and anterior tooth restorations. Amelogen Plus is radiopaque and available in multiple Vita® shades. It is 76% filled by weight and 61% filled by volume and has an average particle size of 0.7µm with narrow upper limit particle distribution. It is supplied in multi-use syringes (2.5 grams) and single compules (0.3 grams)

Directions for Use:

- If refrigerated, bring product to room temperature.
- Apply dentin bonding agent to preparation according to manufacturer's instructions.
- OPTIONAL: Place a super adaptive layer of flowable composite.

For Syringes:

- Remove cap from Amelogen Plus syringe. If desired, use the Quad Key to remove inside flanges for open barrel delivery.
- Twist stem to express composite. Resistance will lessen with each twist.

NOTE: To avoid cross-contamination, use a clean instrument with each increment removed from the syringe.

For Singles:

- Keep tip caps on until use.
- Place syringe compule in dispensing gun.
- Make sure tip is firmly snapped into composite gun before trying to express.

D. Unit dose singles are designed for single use only. Do not re-use to avoid cross-contamination. Discard tips after use.

4. Place Amelogen Plus in incremental layers applying the first layer approximately 0.5 – 1.5mm thick and cure. Follow with 2mm increments curing after each increment.

eaching light. If curing with VALO or another high quality curing light (>600mW/cm2), cure each layer 20 seconds. For lights with an output <600mW/cm2, cure for 40 seconds.

5. After final increment is placed, sculpt and light cure. 6. Shape and define the final anatomy using finishing diamonds or multi-fluted carbide burs. Finish and polish with polishing disks, cups, points or brushes. A composite sealer can be placed over restoration according to manufacturer's instructions to increase surface hardness and reduce surface wear.

Precautions and Warnings:

- Read and understand all instructions before using product.
- For daily use, store at room temperature. Refrigerate bulk storage.
- Use a quality curing light to ensure maximum physical properties of the composite. Uncured material will result in an inferior restoration and may also cause pulpal irritation.
- Clinician and patient should wear UV protective eyewear when curing composite.
- To lessen pre-polymerization of the composite, minimize exposure to bright lights during the procedure.
- To optimize bond to each succeeding layer, do not remove oxygen inhibition layer between incremental layers of composite.
- Ensure composite is totally polymerized between layers and interproximally.
- Syringes are not intended for intraoral use.
- Resins can be sensitizing. Avoid repeated contact of uncured dental resin with skin. Do not use on patients with a known sensitivity to acrylates or other resins. If allergic reaction, dermatitis or rash develops, consult a physician.
- Syringe caps should be immediately replaced after dispensing composite to prevent polymerization from ambient light.
- Disinfect syringes with an intermediate level disinfectant between uses.
- Properly dispose of syringes and compules.
- Keep away from extreme heat and direct sunlight.
- Do not use after expiration date noted on containers.

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