

PermaFlo® Purple Anatomical Indicating Composite

EN

Description:

PermaFlo Purple is a 68% filled, light-cured, radiopaque, thixotropic, methacrylate-based flowable composite with 1µ average particle size.

Indications for Use

For immediate endodontic post-op coronal seal and identification of coronal floor and/or canal orifices.

Directions for Use:

1. Complete endodontic obturation. Clean pulpal floor with cotton and isopropyl alcohol and dry.
2. Apply dentin bonding agent to entire preparation according to manufacturer's instructions.
3. Remove Luer Lock cap and attach syringe tip (Black Mini or Micro 20 gauge)
4. Verify flow prior to applying intraorally.
5. Apply 1-2mm layer of PermaFlo Purple to pulpal floor or over each canal orifice.
6. Light cure with VALO® LED Curing Light on Standard Power for 10 seconds. (For lights with output <600mW/cm2 cure 20 seconds or for lights with output >600mw/cm2 cure for 10 seconds).

Precautions and Warnings:

1. WARNING: Chemical may set in tip after a few hours. To avoid clogging, remove tip and re-cap with Luer Lock caps between each use. Always verify material flow prior to applying intraorally. If resistance is met, replace tip and re-check.
2. Carefully read and understand all instructions before using.
3. Use only recommended tips.
4. Refrigerate PermaFlo Purple when not in use.
5. Resins can be sensitizing. Avoid repeated contact of uncured dental resin with skin. Do not use on patients with a known sensitivity to acrylates or other resins. If allergic reaction, dermatitis or rash develops, consult a physician.
6. Dispose of tips and empty syringes properly.
7. Re-cap syringes with the Luer Lock Cap.
8. Disinfect and wipe syringe with an intermediate level disinfectant between uses or use a disposable syringe cover to avoid cross contamination.
9. Tips are disposable; do not re-use to avoid the risk of cross-contamination.
10. Keep out of reach of children.
11. Do not use after expiration date noted on syringe.
12. For professional use only.

NOTE: For MSDS and additional information about using PermaFlo Purple and other related products, please go to www.ultradent.com

Key:
Legende:
Explication:
Erklärung der Packungssymbole:
Legenda:
Clave:
Legenda:
Teckenförklaring:
Tegnforklaring:
Merkinen selitykset:
Κλειδί:
Обозначения:
Oznaczenia na opakowaniach:
Anhtar:
要點
トピク

See instructions
Siehe Anleitung
Voir le mode d'emploi
Zie gebruiksaanwijzing
Vedere le istruzioni
Vea las Instrucciones
Ver instruçoes
Se bruksanvisning
Se vejledningen
Lue käyttöohjeet
Δείτε τις οδηγίες
Смотрите инструкции
Patrz Instrukcja
Tallmatlára bekkunz
지시사항을 확인하십시오
参照説明
โปรดดูขั้นตอนการใช้

LOT

Batch Code
Liefercode
Code de lot
Partijcode
Código del lotto
Código del lote
Código do lote
Batchkod
Batchkode
Erkãkoodi
Κωδικός παρτίδας
Κωδ παρτίτι
Kod setrii
Seri Kodu
배치 코드
批号
รหัสแบ็ช

2 °C
36 F

8 °C
46 F

Recommended Storage Temperature
Bei °C lagern
Température de stockage recommandée
Aanbevolen bewaar temperatuur
Temperatura di conservazione suggerita
Temperatura de Armazenamento Recomendada
Temperatura de armazenamento recomendada
Rekomenderad förvaringstemperatur
Anbefalet opbevaringstemperatur
Suositeltava säilytyslämpötilä
Συνιστώμενη θερμοκρασία αποθήκευσης
Рекомендуемая температура хранения
Zalecana temperatura przechowywania
Tavsiye edilen Saklama Sıcaklığı
建議室溫儲存
อุณหภูมิการจัดเก็บที่แนะนำ

REF

Use by Date
Verwendbar bis
A utiliser avant
Houdbaarheidsdatum
Usare entro la data indicata
Usar até a data
Användes före
Bruges inden
Käytettävä ennen
Να χρησιμοποιηθείτε μέχρι να
использовано до указанной даты
Oznaczenia na opakowaniach:
Kullanım süresini geçirmeyiniz.
유효기간
有效期限
วันที่หมดอายุการใช้งาน

Catalog Number
Katalognummer
Numéro de catalogue
Catalogusnummer
Numero di catalogo
Número de Catálogo
Número de catálogo
Katalognummer
Katalognummer
Lujelitelänumero
Αριθμός καταλόγου
Номер в каталоге
Numer katalogowy
Katalog Numarası
카탈로그 번호
产品目录编号
หมายเลขแคตตาล็อก

2

Do not reuse to avoid cross contamination
NE/ Nicht wiederverwenden, um Kreuzkontamination zu verhindern.
NE/ Ne pas réutiliser pour éviter la contamination croisée.
N/ Niet opnieuw gebruiken om kruisbesmetting te vermijden.
IT/ Per evitare la contaminazione crociata non riutilizzare.
ES/ No lo reutilice, así evitara la contaminación cruzada.
PT/ Não reutilizar para evitar a contaminação cruzada.
SV/ För undvikande av korskontamination får produkten inte återanvändas.
DA/ Må ikke genbruges for at undgå krydskontaminering.
FI/ Älä käytä uudelleen ristikkontaminaation välttämiseksi.
NO/ Skal ikke brukes om igjen, for å unngå smitteoverføring.
EL/ Μην το επαναχρησιμοποιείτε για να αποφύγετε τη διασταυρωμένη μόλυνση.
RU/ Не использовать повторно во избежание перекрестного загрязнения.
PL/ Używajć jednokrotnie dla uniknięcia zanieczyszczenia krzyżowego
TW/ Carpat kontaminasyonu önlemek için sadece bir kez kullanın.
AR/ لا تعيد استخدامها لتجنب التلوث المتبادل.
KO/ 교차오염을 피하기 위해서는 재사용하지 마십시오.
ZH/ 勿應重複使用，以免交叉感染。
TH/ ห้ามใช้ซ้ำเพื่อหลีกเลี่ยงการปนเปื้อนข้าม

EN- Recycle
DE- Recyclen
FR- Recycler
NL- Recyclen
IT- Riciclare
ES- Reciclar
PT- Reciclar
SV- Återvinna
DA- Genbrüg
FI- Kierrätyskesä
NO- Resirkuler
EL- Ανακυκλώστε το
RU- Утилизировать
PL- Poddać recyklingowi
TR- Geri dönüştürün
AR- أعيد تدويره
KO- 재활용
ZH- 回收使用
TH- รีไซเคิล

Xi

Irritant
Reizmittel
Irritant
Irritierend
Irritante
Irritante
Irritande
Irritante
Irritierende
Irrit
Ärsyttäv ämne
Irriterende stoff
Egeðbrugg
Вызывает раздражения
Związek podrażniający
Tahriş Edici
자극성
刺激性
ฟ้าในระบาศะเคือง

UP Dental GmbH
Am Westhover Berg 30
51149 Cologne Germany

PermaFlo® Purple Anatomical Indicating Composite

Manufactured by:
Ultradent Products Inc
505 West 10200 South
South Jordan, Utah 84095 USA
Made in USA

ULTRADENT
PRODUCTS, INC.
© Copyright 2011 Ultradent Products, Inc. All Rights Reserved. 94805-U.6 021914

Refrigerate if not used on a daily basis

U.S. federal law restricts the sale of this device by or on the order of a dentist.

Keep out of reach of children.

For immediate reorder and/or complete descriptions of Ultradent's product line, refer to Ultradent's catalog or call Toll Free 1-800-552-5512. Outside U.S. call (801) 572-4200 or visit www.ultradent.com.

Ultradent syringes have an expiration date stamped on the side of the syringe consisting of one letter and three numbers. The letter is a lot number used for manufacturing purposes and the three numbers are the expiration date. The first two numbers are the month, and the third number is the last number of the year.

PermaFlo Purple

Flammability

Health

Reactivity

HAZARD RATING
4 = Severe
3 = Serious
2 = Moderate
1 = Slight
0 = Minimal

For MSDS please visit our website: www.ultradent.com