A leader in whitening for over 25 years

A closer look at the rise of Opalescence® – from its ground-breaking launch to its position of dominance in the world of bleaching

Opalescence whitening has gone from being the first and only ADA-accepted, syringe-delivered whitening gel to becoming the number one dentist-recommended professional tooth whitening system in the world! After 26 years, it's still the bright smile standard.

Optident is extremely proud to continue flying the Ultradent flag as a trusted partner in the UK. Since the late 1980s Optident has been a supplier and active educator for the Ultradent range; Opalescence is one of its longest-standing products that has stood the test of time.

Gold standard

Ultradent's Opalescence whitening gel has long been recognised as the 'gold standard' and has been described as 'the most tried and tested tooth whitening product in the world.'

Opalescence was developed and produced by Ultradent in 1990, and is regarded internationally as a product that can satisfy patient's unique whitening needs, which will vary according to lifestyle, budget and overall expectations.

In introducing Opalescence to the dental market, Ultradent's aim was to produce a safe and effective product that would protect teeth from dehydration and also increase shade stability. For these reasons, the product takes the form of an aqueous-based gel that is available in two different levels of carbamide peroxide (10 and 16%), with its formulation incorporating potassium fluoride.



Figure 2a: Tetracycline stained teeth before whitening with Opalescence



Figure 1: The Opalescence 'take home' whitening kit

Potassium fluoride has been found to contribute to an increase in enamel hardness, the prevention of dental caries and a decrease in tooth sensitivity (Basting et al 2003; Al-Qunaian 2005; Clark et al 2006; Amaechi et al 2006; Browning et al 2006).

Opalescence possesses an ideal level of viscosity, helping to hold the tray in place, and prevent any swallowing of the gel, regardless of the method of delivery. A choice of regular, mint or melon flavours is available to suit differing tastes, with overnight wear times for 10% carbamide peroxide of approximately eight to ten hours and four to six hours for a 16% gel. Opalescence is theonly sustained release product on the market. The product can also be highly effective in restoring the



Figure 2b: The same case after whitening with Opalescence

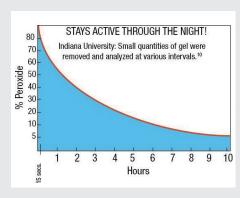


Figure 3: Graph showing the sustained release of fluoride over a 10-hour period

appearance of tetracycline stained teeth, being the preferred treatment method.

Convenient and unique

It is also now available in the unique preloaded Opalescence Go – UltraFit tray which is great for immediate use, post ortho and where patients have a dislike of impression taking.

Opalescence Go is 6% hydrogen peroxide gel, and comes in a convenient and ready to wear tray, that the patient can easily insert and should then suck down to bring into contact with their teeth. The outer layer of the tray is then removed, leaving an inner tray in situ, which should be left in place for approximately 60 minutes each day, with noticeable results











Figures 4a-4e: The ready to use Tres White system. The tray is easily placed, and the outer tray layer is removed, leaving the inner tray in place

being seen in five to ten days.

In seeking to fully satisfy both patient and dental practitioners tooth whitening demands, Optident provides a unique 'bespoke' packaging and design service for

those who wish to create their own unique product promotions for their dental practices. Customised packaging of whitening products featuring practice names, logos and colours









Figure 8: Examples of Optident's bespoke packaging for various private practices. These can be highly effective marketing aids





References

American Dental Association (2010). Tooth whitening /bleaching: Treatment considerations for dentists and their patients. American Dental Association Council on Scientific Affairs. September 2010: 3

Al-Qunaian T (2005). The effect of whitening agents on caries susceptibility of human enamel. Operative Dentistry 30(2): 265-70

Amaechi BT, Clark LM, Barghi N, Summit B (2006). Enamel fluoride uptake from fluoridated carbamide peroxide bleaching gel (Abstract #0498). AADR

Basting RT, Rodrigues A, Serra MC (2003). The effects of seven carbamide peroxide bleaching agents on enamel microhardness over time. JADA 134(10): 1335-42

Browning WD, Myers M, Downey M, Pohjola RM, Brackett WW (2006). Report on low sensitivity whiteners (Abstract #1650). AADR

Clark LM, Barghi N, Summit JB, Amaechi BT (2006). Influence of fluoridated caramide peroxide bleaching gel on enamel demineralization (Abstract #0497). AADR

Haywood VB (1992). History, safety and effectiveness of current bleaching techniques and applications of the nightguard vital bleaching technique. Quintessence International 23(7): 471-88

Haywood VB (2000). A comparison of at home and inoffice bleaching. Dentistry Today. 19(4): 44-53

Haywood VB (2003). Frequently asked questions about bleaching. Compend Contin Educ Dent 24(4A): 324-38

Haywood VB (2008). The 'bottom line' on bleaching. Inside Dentistry 4(2): 82-89

Mokhlis GR, Matis BA, Cochran MA and Eckert GJ (2000). A clinical evaluation of carbamide peroxide and hydrogen peroxide whitening agents during daytime use. JADA 131(9): 1269-1277

Details of all Optident Tooth Whitening products can be obtained from Optident Ltd, International Development Centre, Valley Drive, Ilkley, LS29 8AL.

Tel: 01943 605050 Email: sales@optident.co.uk Website: www.optident.co.uk

