Productnews

The clamp that's making a stamp

Ordering your favourite intraoral clamp just got a whole lot easier. The clamour for the Hallerklammer is on!



Free up hands - the Hallerklammer retracts the cheek and tongue whilst holding cotton rolls and the saliva ejector in place

The highly sought after range of intraoral clamps from Kentzler Kaschner Dental are now available from Optident on British soil. This is good news for the loyal fans out there who have previously struggled to obtain them.

Optident has also won the privilege of introducing the Hallerklammer to the UK - the latest addition to the range specially designed Dr Landenberger. bv Pair this with the black rubber dam, also from Kentzler Kaschner, and you have a perfect combination, providing a fantastic backdrop for photographing your restorative work

Follow the safety instructions - always secure the clamp with floss in one of two ways

Officially known as the Hallerklammer 2A, this multifunctional tool is designed to easily retract the tongue and cheek whilst eliminating moisture contamination. It is ideal for many uses, including Cerec work. Featuring a housing that sits around the molar and two low seated arms positioned either side of the mandible or maxilla, rubber dam or cotton rolls are kept snugly in place.

The upright 'fingers' on the Hallerklammer act as a clip, securing the saliva ejector tube and freeing up hands. Utilising orthodontic bands onto the teeth, on the side of each arm, the gingiva or rubber dam can be retracted; the elastics pull apically against the coronal force of the rubber dam frame.

During second molar restorations when a rubber dam clamp is virtually impossible, clever tricks such as using X-ray film upright in the fingers are a great way to create a tongue barrier.

Colours and sizes

The Hallerklammer can be secured with floss in two ways for the patient's safety and your peace of mind. Place easily with forceps that insert into the top holes adjacent to the ring and release when in place.

The attractive light gold colour option of the Hallerklammer and family is the tougher than steel



zirconium nitride coating. All other colours are a titanium nitride; both are used often in the coating of medical devices such as scalpel blades, bone saw blades and implants. A range of sizes and colours of Hallerklammer clamps are now available from Optident.

The 2A, sometimes referred to as the 'Hallerclamp' and the black rubber dam, is already generating interest among dentists and key opinion leaders of restorative dentistry. They are welcoming the launch of the Hallerklammer and its associated products based on its clever functionality.

Optident is pleased to announce pre-orders have been despatched and products are available to order now. **D**

FOR MORE INFORMATION or to order, contact Optident on 01943 605 050, email sales@optident.co.uk or visit www.optident.co.uk.