

THE BLASTER

by  BIOCLEAR

Bioclear Blaster Care:

1. Please remove the powder cap each evening, or when not in use. **Leaving the cap on and powder in will result in moisture build up within the unit, which may cause damage.**
2. All parts of unit can be heat sterilized. For example; Autoclave, Steamclave, or Statim
3. The Blaster unit can also be wiped in between patients. **There are several wipes and sprays that are causing severe breakdown of plastics on equipment. The warranty on equipment will be void if appropriate sterilization materials are not being used.**

Approved Wipes:

- ProSpray Wipes by Certol
- Optim Wipes by Scican

For more information about wipes & sprays causing damage:

<https://www.gov.uk/drug-device-alerts/medical-device-alert-detergent-and-disinfectant-wipes-used-on-reusable-medical-devices-with-plastic-surfaces-risk-of-degrading-plastic-surfaces>