# QUICK GUIDE

AIRFLOW PROPHYLAXIS MASTER®

"I FEEL GOOD"

**EMS+**  
MAKE ME SMILE.

# DOWNLOAD ON MYEMS.DENTAL

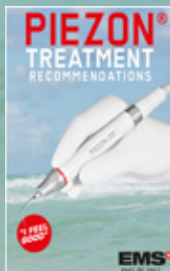
General Instructions:  
installation, cleaning,  
reprocessing, maintenance ...

Treatment Recommendations  
Guided Biofilm Therapy

Extended use  
of PIEZON®  
instruments for  
Endodontology  
and other  
dental  
applications.

EMS Dental  
Installation & Maintenance

# SUMMARY

|               |      |                |       |
|---------------|------|----------------|-------|
| EN ENGLISH    | p.4  | SK SLOVENSKY   | p.60  |
| DE DEUTSCH    | p.8  | GR ΕΛΛΗΝΙΚΑ    | p.64  |
| FR FRANÇAIS   | p.12 | RO ROMANIAN    | p.68  |
| IT ITALIANO   | p.16 | HU MAGYAR      | p.72  |
| ES ESPAÑOL    | p.20 | TR TÜRKÇE      | p.76  |
| PT PORTUGUÊS  | p.24 | ET EESTI KEEL  | p.80  |
| NO NORSK      | p.28 | LV LATVIEŠU    | p.84  |
| SE SVENSKA    | p.32 | LT LIETUVIŠKAI | p.88  |
| FI SUOMI      | p.36 | RU РУССКИЙ     | p.92  |
| NL NEDERLANDS | p.40 | ID INDONESIA   | p.96  |
| DK DANSK      | p.44 | TH ไทย         | p.100 |
| PL POLSKI     | p.48 | VN TIẾNG VIỆT  | p.104 |
| CZ ČESKÝ      | p.52 | CN 英文          | p.108 |
| SI SLOVENSKO  | p.56 | JP 日本語         | p.112 |

# WELCOME

EN

We thank you for your trust in our newest product AIRFLOW® Prophylaxis Master, which will guide you into the world of our GBT - GUIDED BIOFILM THERAPY.

Register your new unit on [www.myems.dental](http://www.myems.dental) and get an extra year of warranty. This warranty is only valid if you have completed your first yearly maintenance within 12 months of purchase.

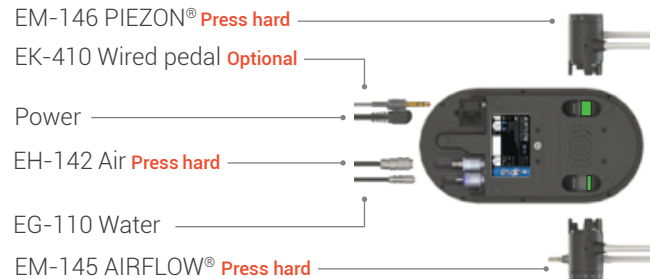
Before installation, download & read complete instructions for use.


## Connections

- + EH-142 Air hose to the cabinet / dental unit.
- + EG-110 Water hose to the cabinet / dental unit.

Check cords are locked.

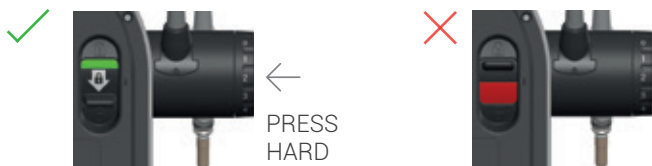Install the batteries in the pedal.

EN

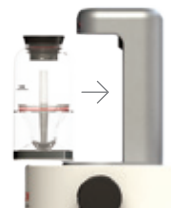
**PLUS** powder: Sub and Supragingival.

**CLASSIC** powder: Supragingival on enamel only.

Insert the powder chamber.

The use of the PIEZON® Bottle is optional.

If the bottle is not placed onto the device then you are connected to the external water supply.

Functions

Hereby, Electro Medical Systems S.A., declares that this Medical Device and all of its accessories having radio equipment are compliant with the European Directive 2014/53/EU (RED Radio Equipment Directive). The full text of the EU declaration of conformity is available in the Instruction for Use.

# CLEAN

EN

## EACH MORNING Rinse

## BETWEEN Each patient

## END OF DAY Clean

# MAINTENANCE

EN

## CHECK IT

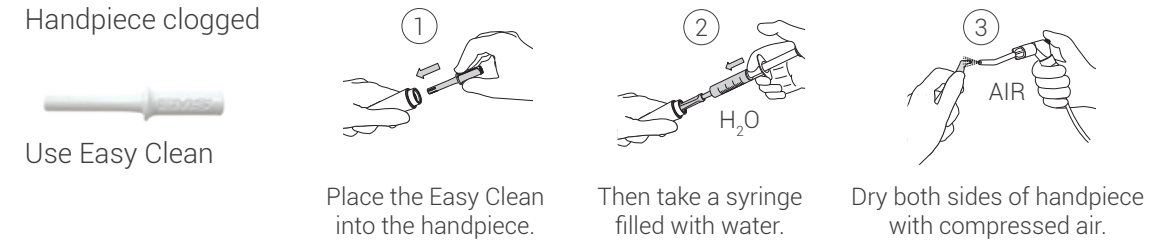
## EACH YEAR Preventive maintenance

Send your device to your authorized EMS repair center.

## TROUBLESHOOTING

Handpiece clogged


Service needed  
Stop using your device.

Missing cord  
Check your cords connection.

Batteries low  
Change pedal batteries.

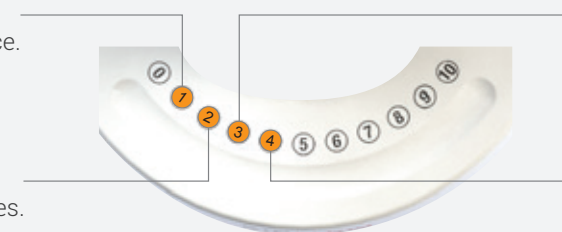Overheating  
Wait for cooling.  
If error repeats, call service.

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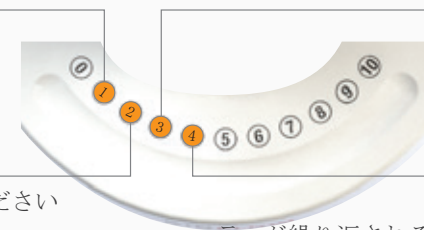
**+** WORLDWIDE EMS AFFILIATES

DE

**MUNICH, GERMANY****EMS ELECTRO MEDICAL SYSTEMS GMBH**

Schatzbogen 86

D-81829 München

Tel. +49 89 42 71 61 0

Fax +49 89 42 71 61 60

e-mail: [info@ems-ch.de](mailto:info@ems-ch.de)

FR

**PARIS, FRANCE****EMS ELECTRO MEDICAL SYSTEMS FRANCE SARL**

23, avenue Louis Breguet

Immeuble Santos Dumont, Batiment D

F-78140 Velizy Villacoublay - France

Tel. +33 1 34 58 03 80

e-mail: [info@ems-france.fr](mailto:info@ems-france.fr)

ES

**MADRID, SPAIN****EMS ELECTRO MEDICAL SYSTEMS ESPAÑA SL**

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**E.M.S. ELECTRO MEDICAL SYSTEMS TRADING (SHANGHAI) CO., LTD.**

24A, Jin Sui Mansion,

N°379 Pudong Nan Rd.

Shanghai, China 200120

Tel. +86 21 3363 2323

e-mail: [emschina@ems-ch.com](mailto:emschina@ems-ch.com)

US

**DALLAS, USA****EMS Corporation**

11886 Greenville Avenue, Ste 120

Dallas, Texas 75243

United States of America

Phone : +001 972 690 8382

Fax : +001 972 690 8981

E-mail : [info@ems-na.com](mailto:info@ems-na.com)

JP

**TOKYO, JAPAN****E.M.S. JAPAN BRANCH OFFICE**

501, 73 Kanda Neribeicho Chiyoda-ku

Tokyo 101-0022 - Japan

Phone : +81 (0) 3 5207 6795

Fax : +81 (0) 3 5207 6796

E-mail : [emsjapan@ems-ch.com](mailto:emsjapan@ems-ch.com)

# FOLLOW US

EMS Dental

EMS Dental -  
International

EMS Dental


No. 01-17-416359 - www.myclimate.org  
© myclimate - The Climate Protection Partnership

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# ONLY USE ORIGINAL EMS

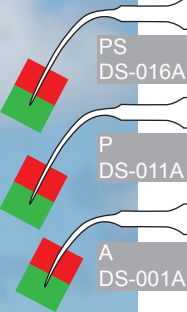Check and renew regularly your  
PIEZON® instrument.

Only use original AIRFLOW®  
powders

CE 0124

**EMS** 

PRINTED IN SWITZERLAND  
[WWW.EMS-DENTAL.COM](http://WWW.EMS-DENTAL.COM)