## UltraSeal XT® hydro™

Pit & Fissure Sealant

Description:

UltraSeal XT® hydro™ is a 53% filled, light-cure, radiopac, methacrylate-based, thixotropic resin sealant. It is hydrophilic and has a self-adhesive quality. UltraSeal XT® hydro™ chemistry provides the option of a visual verification for marginal retention with the use of a UV light, upon placement and at recall visits. The UV light is not included in the UltraSeal XT® hydro™ kit.

Indications for Use: Use UltraSeal XT® hydro™ for prophylactic sealing of pits and fissures.

Directions for Use

Pit and fissure sealant

1. Attach Inspiral® Brush tip or Blue Micro® tip to Ultra-Etch® 35%.

2. Attach Inspiral® Brush tip to UltraSeal XT® hydro™ syringe.

3. Using an air abrasion unit, a thin dental fissure/prep bur, air polishing prophylaxis system, a mixture of water and flour of pumice, Consensus Scrub® or UltraPro® Tx Pure Prophy Paste,

clean fissures.

NOTE: UltraPro® Tx Pure Prophy Paste or flour of pumice is recommended over standard prophylaxis paste because it easily rinses from the fissures and will not obstruct the fissure.

4. Isolate teeth being treated to avoid saliva contamination.

5. Apply etch to fissures for 2 seconds. (Figure 1). Rinse etch off tooth.

6. Lightly air dry the tooth to remove standing/pooled/puddled water. Tooth can be moist. Do not desiccate the tooth.

NOTE: If using sodium bicarbonate in air polishing system, repeat the etch/rinse step.

7. Express a small drop of UltraSeal XT® hydro™ to the brush tip prior to applying intraorally. (Figure 2)

8. Using a painting action followed by a light agitation with the brush tips, apply resin to deep fissures. Avoid pooling resin. (Figure 3)

9. Light cure with VALO® LED Curing Light on Standard Power for 20 seconds or 3 seconds on Xtra Power mode. (For lights with output <600mW/cm² cure 40 seconds or for lights with output >600mW/cm² cure for 20 seconds). (Figure 4)

10. Check margins and adjust occlusion, if necessary.

NOTE: As an option, UV light may be used for visual verification of marginal retention. Either the VALO® Curing Light with the BlackLight Lens specifically designed to filter for UV light, or a small UV light, is available through Ultradent. For best results, redirect overhead dental light when using UV light.

Precautions and Warnings:

1. Carefully read and understand all instructions before using UltraSeal XT® hydro™.

2. Clinician and patient should wear UV protective eyewear when curing sealant.

3. When using VALO® Curing Light, the VALO® with the VALO® BlackLight Lens or any UV light, use caution not to shine the light directly toward the eyes.

4. Verify flow of all syringes prior to applying intraorally.

If resistance is met, replace tip and re-check. Use only recommended tips.

5. Store at room temperature.

6. Redirect overhead light to prevent premature polymerization.

7. Resins can be sensitizing. Avoid repeated contact of uncured dental resin with skin. Do not use on patients with a known sensitivity to acrylates or other resins.

If allergic reaction, dermatitis or rash develops, consult a physician.

8. Re-cap immediately following use to avoid polymerization in tip and syringe.

9. Dispose of tips and syringes properly.

10. Keep out of reach of children.

11. Use Ultradent Syringe Covers and/or clean and disinfect syringes between patients.

NOTE: For SDS and additional information about using UltraSeal XT® hydro™ and other related products, please go to [www.ultradent.com](http://www.ultradent.com).

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