

PROTECT YOURSELF against needle stick injury

CONTA-GUARD™ protects you against accidental needle sticks during the entire process – from preparation to disposal.

CONTA-GUARD™ laminated disposable shield seals the rear part of the needle. Once the contaminated needle has been safely sealed, all further handling is absolutely safe.

The CONTA-GUARD™ Stand permits safe one-handed de- and recapping of the needle, also when repeated injection to the same patient is needed.

The system can be used with all standard dental syringes and needles.

It is easy to handle and to integrate into the daily routines.



Remove the perforated field (pull-out opening) on the side/bottom of the box.



Place the prepared syringe in the stand. Then press the wings of the stand slightly. Hereby the needle cap is retained in the stand, so the syringe/needle can be lifted free of the needle cap.



After injection: by one hand place the syringe in the stand to re-cap the needle. Be careful not to penetrate the needle cap with the sharp needle tip.



Fold the CONTA-GUARD™ firmly around the rear part of the needle.



Remove the protective film. Press the needle cap through the hole in the CONTA-GUARDTM, with the adhesive side facing up.



The needle cap with CONTA-GUARD™ is left in the stand when injection takes place.



Lift up the syringe from the stand and unscrew the needle.



The needle is now sealed and you are completely protected against needle stick injury.



View animated step by step-guide



CONTA-GUARD™

An efficient protection against needlestick injury. It protects during the entire process - from preparation to disposal of the contaminated needle.

CONTA-GUARD™

Laminated disposable shields, box of 200 pcs., RØNVIG art. no. 2200 Re-capping stand, RØNVIG art. no. 2222 Designed and manufactured in Denmark by



Gl. Vejlevej 59 · DK-8721 Daugaard · Tel.: +45 70 23 34 11 Fax: +45 76 74 07 98 · E-mail: export@ronvig.com · www.ronvig.com