QUICK GUIDE

AIRFLOW ONE®

"I FEEL GOOD"

EMS⁺
MAKE ME SMILE.

DOWNLOAD ON MYEMS.DENTAL

General Instructions:
installation, cleaning,
reprocessing, maintenance ...

Treatment Recommendations
Guided Biofilm Therapy

 EMS Dental
Installation & Maintenance

SUMMARY

EN	ENGLISH	p.4	SK	SLOVENSKY	p.60
DE	DEUTSCH	p.8	GR	ΕΛΛΗΝΙΚΑ	p.64
FR	FRANÇAIS	p.12	RO	ROMANIAN	p.68
IT	ITALIANO	p.16	HU	MAGYAR	p.72
ES	ESPAÑOL	p.20	TR	TÜRKÇE	p.76
PT	PORTUGUÊS	p.24	ET	EESTI KEEL	p.80
NO	NORSK	p.28	LV	LATVIEŠU	p.84
SE	SVENSKA	p.32	LT	LIETUVIŠKAI	p.88
FI	SUOMI	p.36	RU	РУССКИЙ	p.92
NL	NEDERLANDS	p.40	ID	INDONESIA	p.96
DK	DANSK	p.44	TH	ไทย	p.100
PL	POLSKI	p.48	VN	TIẾNG VIỆT	p.104
CZ	ČESKÝ	p.52	CN	英文	p.108
SI	SLOVENSKO	p.56			

WELCOME

EN

We thank you for your trust in our newest product AIRFLOW® ONE, which will guide you into the world of our GBT - GUIDED BIOFILM THERAPY.

Register your new unit on www.myems.dental and get an extra year of warranty. This warranty is only valid if you have completed your first yearly maintenance within 12 months of purchase.

Before installation, download & read complete instructions for use.

Connections

- + EH-142 Air hose to the cabinet / dental unit.
- + EG-110 Water hose to the cabinet / dental unit.


EK-410 Wired pedal **Optional**

Power

EH-142 Air **Press hard**

EG-110 Water

EM-145 AIRFLOW® **Press hard**

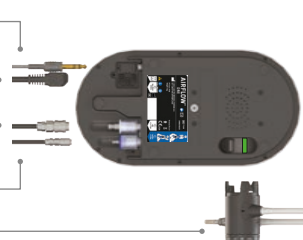Check cord is locked.

PRESS HARD

Install the batteries in the pedal.

PLUS Powder: Sub and Supragingival.

CLASSIC Powder: Supragingival on enamel only.

Insert the powder chamber.

The use of the WATER Bottle is optional.

If the bottle is not placed onto the device then you are connected to the external water supply.

Functions

AIRFLOW® pressure

ON/OFF

Power set

NORMAL **Press gently**

BOOST **Press hard**

AIRFLOW® water

Hereby, Electro Medical Systems S.A., declares that this Medical Device and all of its accessories having radio equipment are compliant with the European Directive 2014/53/EU (RED Radio Equipment Directive). The full text of the EU declaration of conformity is available in the Instruction for Use.

CLEAN

EN

EACH MORNING Rinse

Place your water bottle onto the device.

Set water to 10, Turn the device ON.

Hold the cord above the sink.

Press the pedal during 30 seconds.

BETWEEN Each patient

Clean the outside surface of the device with a cloth and alcohol.

Reprocess your Handpieces.

END OF DAY Clean

Place your NIGHT CLEANER® bottle onto the device.

Set water to 10. Turn the device ON.

Hold the cord above the sink.

Press the pedal during 30 seconds.

MAINTENANCE

EN

CHECK IT

Check the blue water filter and change it if needed.

EACH YEAR Preventive maintenance


Send your device to your authorized EMS repair center.

TROUBLESHOOTING


Handpiece clogged


Use Easy Clean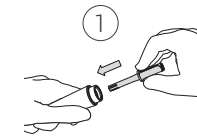

Place the Easy Clean into the handpiece.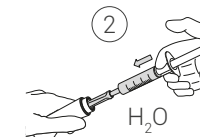


Then take a syringe filled with water.

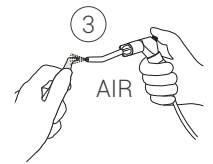Dry both sides of handpiece with compressed air.

Service needed
Stop using your device.

Batteries low
Change pedal batteries.

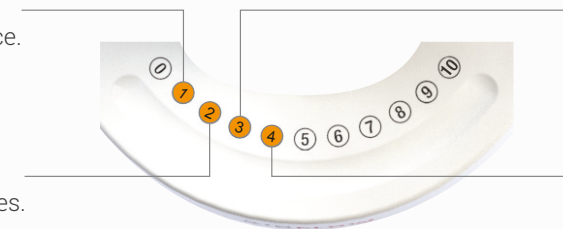Missing cord
Check your cord connection.

Overheating
Wait for cooling.
If error repeats, call service.

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EMS Corporation