# Cleaning, disinfection & sterilization guide

To ensure safe and optimal use of  
Polydentia's medical device

## Index

EN - English	2	SE - Svenska	29
DE - Deutsch	5	DK - Dansk	32
FR - Français	8	PL - Polski	35
IT - Italiano	11	CZ - Čeština	38
ES - Español	14	TK - Türkçe	41
PT - Português	17	GR - ελληνικά	44
NL - Nederlands	20	JP - 日本語	47
NO - Norsk	23	CN - 中国	50
FI - Suomi	26	AR - العربية	53

## Cleaning, disinfection and sterilization procedures for Polydentia’s medical devices

The table below identifies which Cleaning, disinfection and sterilization procedure shall be followed for each Polydentia’s Medical Device requiring a reprocessing. A comprehensive description of each procedure is reported in the following pages.

Medical Device	REF (Polydentia article number)						Procedure 1	Procedure 2
myClip 2.0	6305	6805					X	
myRing Forte	6803	6806					X	
myRing Classico	6801	6802	6804				X	
Tines for myClip 2.0 & myRing Forte	6305	6803	6805	6806	6303			X
Delta Tubes	6801	6802	6804	5712				X
myForceps	6803	5707						X
myQuickmatrix Forceps	5703							X
Maximat Plus tensioning instrument	5190							X
Maximat Plus autoclavable spool	5190	5127						X
Clip & Splint	5980	5990	5970	5971	5979	5975CS		X
Vistatec Autoclavable Frame	5613							X
Hy-Drop	5411/Y	5411/O	5411/G	5411/V	5411/B			X
HyDrop Forte	5412							X
Hy-Drop, Membranes	5411/Y	5411/O	5411/G	5411/V	5411/B	5412	5415	X
Blue PolyShapes	5960							X

## Procedure 1

### Cleaning and Sterilization Guide for multi-part medical devices

#### Preliminary operations:

- Polydentia's products are always supplied non-sterile.  
For safe and optimal use always clean, disinfect and sterilize your product prior to use
- Check and inspect your equipment regularly.  
Always refer to manufacturer's instructions when using your equipment
- Please note that regulations may vary depending on the country where you are living.  
Always refer to the regulation applicable in your country
- Materials may alter over time. Sterilization or exposure to chemicals can accelerate this deterioration.  
Always check your product for wear and damage before use
- Avoid contact of dissimilar metals at any point during the cleaning and sterilization process

#### Cleaning:

**Products should be cleaned as soon as possible after use to prevent soils from hardening**

1. Disassemble all parts of the medical device
2. Remove visible debris or organic matter with a disposable wipe or surface brush.  
This has to be done by scrubbing with warm tap water and soap or detergent
3. Inspect instruments for debris on surfaces. Repeat cleaning step 2 if there are still debris
4. Immediately soak in an appropriate disinfecting solution, following the manufacturer's instructions
5. Alternatively to point 4, an ultrasonic bath cleaner with an appropriate cleaning and disinfecting agent can be used, in accordance with the manufacturer's instructions. Please assure that the medical device has the relevant symbol on the packaging
6. Rinse all parts with warm tap water
7. Alternatively to points 2 to 6, a thermo disinfection with an appropriate cleaning agent can be used in accordance with the manufacturer's instructions. Please assure that the medical device has the relevant symbol on the packaging
8. Dry all the medical device components, then place them in an autoclave tray, or in a sterilization pouch (see Sterilization instructions step 2 below)

#### Sterilization:

1. Sterilization is not a substitute for proper cleaning. Clean the instrument first (see section Cleaning) and afterwards sterilize it
2. Place all the medical device components in an autoclave tray, or in a sterilization pouch
3. Sterilize in a steam autoclave using distilled water and at the temperature specified on the relevant symbol
4. Do not use chemical autoclave or cold sterilization solutions
5. Avoid direct contact with the metallic part of the autoclave

#### IMPORTANT:

The medical device always has to be disassembled before reprocessing (cleaning, packaging, sterilization and storage).

#### DISCLAIMER:

This high quality product is manufactured by Polydentia SA, in Mezzovico, Switzerland. The content of this packaging has been inspected, tested and approved by our quality officers.

Materials used during our manufacturing process are designed to provide durability and performance. Polydentia constantly looks for the highest quality plastics for its product; nevertheless please take into account that plastics may alter over time. Sterilization or exposure to chemicals can accelerate this deterioration.  
For the longest lifetime of your premium quality product always clean and sterilize it following these instructions.

## Procedure 2

### Cleaning and Sterilization Guide for medical devices

#### Preliminary operations:

- Polydentia's products are always supplied non-sterile. For safe and optimal use always clean, disinfect and sterilize your product prior to use
- Check and inspect your equipment regularly. Always refer to manufacturer's instructions when using your equipment
- Please note that regulations may vary depending on the country where you are living. Always refer to the regulation applicable in your country
- Materials may alter over time. Sterilization or exposure to chemicals can accelerate this deterioration. Always check your product for wear and damage before use
- Avoid contact of dissimilar metals at any point during the cleaning and sterilization process

#### Cleaning:

**Products should be cleaned as soon as possible after use to prevent soils from hardening**

1. Remove visible debris or organic matter with a disposable wipe or surface brush. This has to be done by scrubbing with warm tap water and soap or detergent
2. Inspect instruments for debris on surfaces. Repeat cleaning step 2 if there are still debris
3. Immediately soak in an appropriate disinfecting solution, following the manufacturer's instructions
4. Alternatively to point 4, an ultrasonic bath cleaner with an appropriate cleaning and disinfecting agent can be used, in accordance with the manufacturer's instructions. Please assure that the medical device has the relevant symbol on the packaging
5. Rinse the medical device with warm tap water
6. Alternatively to points 2 to 6, a thermo disinfection with an appropriate cleaning agent can be used in accordance with the manufacturer's instructions. Please assure that the medical device has the relevant symbol on the packaging
7. Dry the medical device, then place them in an autoclave tray, or in a sterilization pouch (see Sterilization instructions step 2 below)

#### Sterilization:

1. Sterilization is not a substitute for proper cleaning. Clean the instrument first (see section Cleaning) and afterwards sterilize it
2. Place the medical device in an autoclave tray, or in a sterilization pouch
3. Sterilize in a steam autoclave using distilled water and at the temperature specified on the relevant symbol
4. Do not use chemical autoclave or cold sterilization solutions
5. Avoid direct contact with the metallic part of the autoclave

#### DISCLAIMER:

This high quality product is manufactured by Polydentia SA, in Mezzovico, Switzerland. The content of this packaging has been inspected, tested and approved by our quality officers.

Materials used during our manufacturing process are designed to provide durability and performance. Polydentia constantly looks for the highest quality plastics for its product; nevertheless please take into account that plastics may alter over time. Sterilization or exposure to chemicals can accelerate this deterioration. For the longest lifetime of your premium quality product always clean and sterilize it following these instructions.

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