

Bisco

ZirClean

Restoration Cleaner

Reinigungsmittel für Restaurationen
Nettoyant pour restaurations prothétiques
Gel limpiador de superficie de restauración
Gel per la Pulizia dei Restauri
Rengöringsmedel för ersättningar
Rengøringsmiddel til fast protetik
Restaaraatioiden puhdistusgeeli
Restauratiereiniger
Rengjøring av restaureringen
Agente de Limpeza para Restaurações
Καθαριστικό αποκαταστάσεων
материал для очистки ортопедических
реставраций
Żel do oczyszczania prac protetycznych

INTC-075

REF B-7351P

**Contents: 1 Syringe ZirClean (5), Accessories,
Instructions**



Single Use
Accessories



For Dental
Use Only



Consult
Instructions
For Use



Recommended
Storage Temp.




BISCO, Inc.
1100 W. Irving Park Rd.
Schaumburg, IL U.S.A.
1-847-534-6000
www.bisco.com



ZirClean




1. Try in (to check occlusion and fitting).
2. After try-in, thoroughly rinse the restoration with water spray and dry with oil-free air.

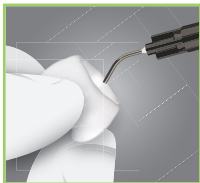

3. Cover all bonded surfaces of the restoration with a layer of ZirClean. Tips are single use only; make sure to dispose of them properly before using material again.
4. Allow 20 seconds for the cleaning action of ZirClean to take effect, then thoroughly rinse with water spray and dry with oil-free air.

* Sandblast internal surface of restoration if necessary.

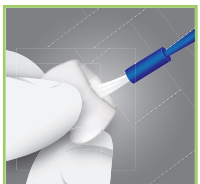

5. Prime the bonding surface of the restoration with a suitable primer (such as Z-PRIME™ Plus* or PORCELAIN PRIMER*) according to manufacturer's instructions.



Consult
Instructions
For Use