



ACTEON

moving forward

AIR N GO

EASY



USING AND MAINTAINING YOUR UNIT

USES

Supra cleaning

- Use the supra nozzle and sodium bicarbonate classic powder. This powder is made of soft 76µm molecules and is perfect for all types of stain removal.
- Do not point the nozzle at the soft tissue. The powder will not damage soft tissue, but the pressure of the spray is 3 bar and can be uncomfortable if directed at the gum or lips.
- Top-up the powder as required during treatment. Take care not to pour it into the inner metal tube.
- Do not re-use powder.



Subgingival cleaning

- Use the perio nozzle (shoehorn fl at ended shape does up to 8mm pockets, nozzle with laser markings does up to 10mm pockets)
- Only work sub-gingivally with the perio powder, NOT the classic.
- Perio powder is Glycine and 25 µm molecule size. It does not damage the soft tissue.
- Clean each surface of the root for 5 seconds.
- Use for the following applications:
 - Biofilm disruption & PMPR .
 - Implant treatment & maintenance, .
 - Periodontal treatment & maintenance.




MAINTENANCE

- 1 Leave the unit attached, remove the lid of the tank and empty the powder.
- 2 Replace the lid and run the unit on the delivery system to flush out any residual powder.
- 3 Use the metal cleaning sticks and 3ml syringe with

yellow flexible tip, to flush through the nozzles and centre part of the unit.

- 4 Dry the unit completely, especially the powder tank.
- 5 Sterilise the nozzle and outer sleeve.
- 6 Check and lubricate the O'rings regularly. Please refer to the diagram provided.

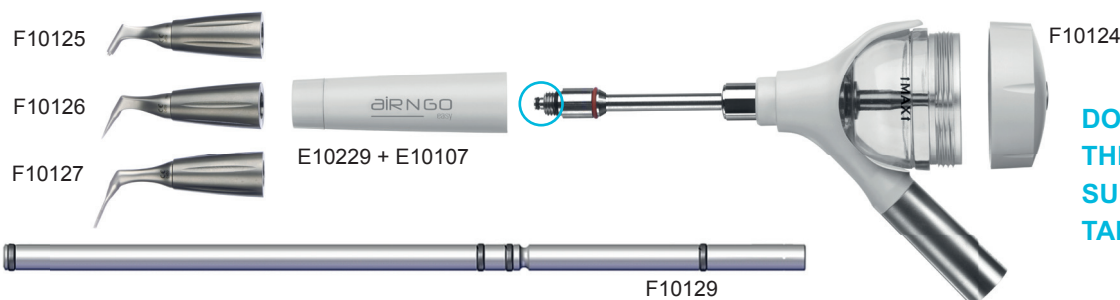

DO NOT AUTOCLAVE THE WHOLE UNIT OR SUBMERGE THE POWDER TANK IN HOT WATER

HINTS & TIPS

We strongly recommend you purchase at least one spare nozzle and inner retrofit metal tube in case of a blockage. To order a new inner tube quote reference F10129 to your preferred supplier.

You can remove and insert this part by pulling it from the end marked with a blue circle, or from the powder tank end. In the event of a blockage, this part along with the nozzle, can be sent to our repair team: Acteon Repairs, 1-2 Apollo Court, Hallam Way, Whitehills Business Park, FY4 5FS. The complete device does not need to be returned. You can request a free post jiffy bag by emailing your address to info.uk@acteongroup.com.

If the silver button on the cap of the tank can be depressed and released, please request a new unmoveable version under warranty by emailing info.uk@acteongroup.com or calling 0800 038 9840 option 1.