



ACTEON

moving forward

AIR N GO

EASY

USES

Supra cleaning

- Use the supra nozzle and sodium bicarbonate classic powder. This powder is made of very soft 76µm molecules.
- Use for biofilm disruption, PMPR and stain removal.
- Do not point the nozzle at the soft tissues. The powder will not cause damage but the pressure of the spray is 3 bar and can be uncomfortable if directed at the gum or lips.
- Top-up the powder as required during treatment. Take care not to pour it into the inner metal tube.



Subgingival cleaning

- Use the perio nozzle and Glycine perio powder.
 - *The shoehorn flat ended nozzle will go up to 8mm.
 - *The laser marked nozzle will go up to 10mm.
- Only work sub-gingivally with the perio powder, NOT the classic.
- The perio powder is 25 µm molecules. It does not damage the soft tissue.
- Clean each surface of the root for 5 seconds.
- Use for the following applications:
 - Biofilm disruption & PMPR .
 - Implant treatment & maintenance, .
 - Periodontal treatment & maintenance.



MAINTENANCE

It is essential to incorporate this maintenance routine between every patient.

- 1 Leave the unit attached to the hose, remove the lid of the tank and empty the powder.
- 2 Replace the lid and run the unit on the delivery system to flush out any residual powder.
- 3 Use the cleaning probe to clean through the inner retrofit tube.
- 4 Use a k-flex endo file to clean the nozzle.
- 5 Dry the unit completely, especially the powder tank.
- 6 Sterilise the nozzle and outer sleeve.
- 7 Check and lubricate the o'rings regularly. Please refer to the diagram provided with the unit.



DO NOT AUTOCLAVE THE WHOLE UNIT OR SUBMERGE THE POWDER TANK IN HOT WATER

TIPS

We strongly recommend you purchase at least one spare nozzle and inner retrofit metal tube.

Do not re-use any powder.

Important codes (quote to your preferred supplier when reordering):

Inner retrofit tube	F10129
Long cleaning probe	F10208
End cap for tank	F10124
O'ring kit with grease and diagram	F10205

You can remove and insert the inner retrofit tube by pulling it from the end marked with a blue circle in the diagram above, or from inside the powder tank. In the event of a blockage, this part along with the nozzle (if blocked) can be sent to the repair team: Acteon Repairs, 1-2 Apollo Court, Hallam Way, Whitehills Business Park, Blackpool, FY4 5FS. You can request a free post returns label by emailing info.uk@acteongroup.com.