



USING AND MAINTAINING YOUR UNIT

USES

Supra cleaning

- Use the supra nozzle and sodium bicarbonate classic powder. This powder is made of soft 76µm molecules and is perfect for all types of stain removal.
- Do not point the nozzle at the soft tissue. The powder will not damage soft tissue, but the pressure of the spray is 3 bar and can be uncomfortable if directed at the gum or lips.
- A full tank of powder will do a full mouth of heavy stain.
- Do not re-use powder.



Subgingival cleaning

- Use the perio nozzle (shoehorn flat ended shape does up to 8mm pockets, nozzle with laser markings does up to 10mm pockets)
- Only work sub-gingivally with the perio powder NOT the classic.
- Perio powder is Glycine and 25 µm molecule size. It does not damage the soft tissue.
- Clean each surface of the root for 5 seconds.
- Use for the following applications:
biofilm disruption,
implant maintenance,
periodontal disease.



MAINTENANCE

After each treatment

- Leave the unit attached, remove the lid of the tank and empty the powder.
- With the lid on, run the unit on the delivery system to flush out any residual powder.
- Ensure the powder tank is completely dry.
- Sterilise the nozzle and outer sleeve.

At the end of a morning and evening session

- Use the metal cleaning sticks and 3ml syringe with yellow flexible tip, to flush through the nozzles and centre part of the unit.
- Dry the unit completely.
- Ensure the o'rings are present and lubricate if necessary.

