


I AM VERSATILE

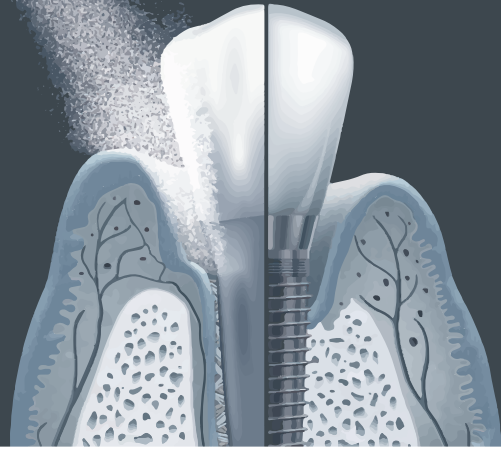

airNGO
easy



**All clinical applications
are easy with the
all-in-one air polisher**

ACTEON

SIMPLE AIR POLISHING

airN GO
easy

DUAL FUNCTIONS

- **"SUPRA"** function" for supragingival polishing and stain removal
- **"PERIO"** function" for subgingival polishing, periodontal maintenance and peri-implant treatments



A single handpiece to cover a range of care and treatment
AIR-N-GO® easy combines essential functions and reliability:

- Fast start-up: direct connection to the high speed handpiece connector
- 360° swivel handpiece for fast and accurate treatment
- Parts can be dismantled for easy cleaning

4 Nozzles for Treatment

- Sodium bicarbonate **"CLASSIC"** powders
- Calcium carbonate **"PEARL"** powder
- Glycine **"PERIO"** powder



A COMPLETE SOLUTION FOR IMPLANT MAINTENANCE



NEWTRON®



NEWTRON



implantProtect
Pure Titanium

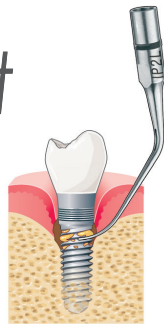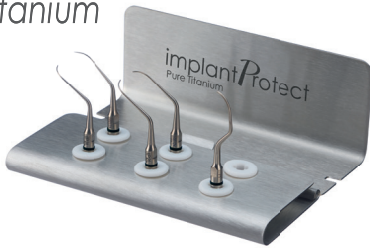

AIR-N-GO
easy

IMPLANT PROTECT TIP KIT

AIR-N-GO® EASY + PERIO EASY KIT

DETECT
DIAGNOSE
TREAT

Contact ACTEON for a Detailed Implant Maintenance Protocol

PREVENT & FIGHT DISEASE

AIR-N-GO® nozzles provide continuous, accurate and controlled spray. These nozzles offer power and efficiency for the removal of supra and subgingival biofilm.

AIR-N-GO® SUPRA NOZZLE

(standard)



Clinical case Dr. P. ROUAS (France)

- Suitable for any supra-gingival prophylactic treatment in combination with **AIR-N-GO®** "CLASSIC" or "PEARL" powders

AIR-N-GO® PERIO MAINTENANCE NOZZLE

(optional)



Clinical case Dr. C. LIM (U.K.)

- Reduced consumption of **AIR-N-GO®** "PERIO" fine powder particles supragingivally
- For use on implants and appliances

AIR-N-GO® PERIO easy NOZZLE

(optional Perio easy kit)



Clinical case Dr. C. LIM (U.K.)

- Innovative design in a "shoehorn" shape providing easy sub-gingival insertion
- For non-surgical, sub-gingival treatment of teeth and implants

AIR-N-GO® PERIO NOZZLE

(optional)



Clinical case Dr. A. PATEL (U.K.)

- Sub-gingival use for surgical and non-surgical treatments (tunnelling and open flaps)
- Powerful double lateral spray
- For use on implants



CUSTOMIZED SOLUTIONS FOR YOUR CLINICAL NEEDS

airN GO
powder

AIR-N-GO® "CLASSIC" POWDERS

(Sodium bicarbonate)
(≈ 76µm average grain size)

Complete range of active supragingival powders

- Cleaning provides instant results
- Remove stains and bacterial plaque, even in areas that are difficult to access such as pits and fissures
- Better patient comfort: gentle and efficient with limited bleeding during treatment




**5 fresh flavors based on 100% natural aromas or essential oils:
Neutral, Cola, Raspberry, Peppermint and Lemon**

AIR-N-GO® "PEARL" POWDER



SODIUM FREE!

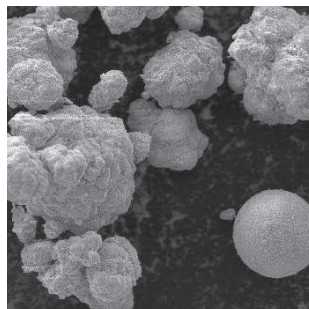

AIR-N-GO® "PEARL" (x1000)
Source: ICMCB CNRS (Institute of Condensed Matter Chemistry - French National Centre for Scientific Research) S.E.M. photos

*Natural calcium carbonate-based
(≈ 55µm average grain size)*


Supragingival polishing powder for delicate tissue protection

- Reduction in pH acidity caused by tissue inflammation; return to clinically healthy buccal flora
- Restoration of the natural shine and pearliness of enamel
- Fast cleaning of difficult stains and composites

AIR-N-GO® "PERIO" POWDER



SODIUM FREE!

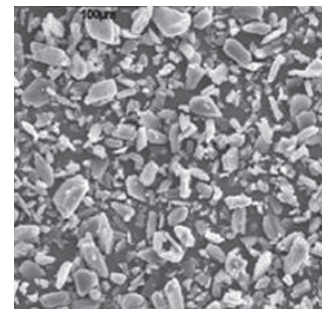

AIR-N-GO® "PERIO" (x1000)
Source: ICMCB CNRS (Institute of Condensed Matter Chemistry - French National Centre for Scientific Research) S.E.M. photos

*Natural glycine-based
(≈ 25µm average grain size)*

- Low abrasiveness preserving the treated surfaces and the most fragile anatomical areas
- Clinically proven to decrease biofilm subgingivally
- Safe to use on any implant surface

air N GO








easy



GO SUPRA

Teeth and Implants

GO PERIO

Nozzles	Powders	Indications
 <p>AIR-N-GO® easy SUPRA 120° Supplied as standard</p>	 <p>"CLASSIC" Sodium bicarbonate-based Average particle size: 76µm FLAVORS: Neutral, Cola, Raspberry, Peppermint and Lemon</p>	<ul style="list-style-type: none"> • Finishing after scaling • Preparation for whitening • Cleaning of interproximal areas, fissures and crevices • Preparation of the tooth surface before etching • Cleaning of orthodontic brackets • Removal of supragingival biofilm, plaque, stains and temporary cement residues
	 <p>"PEARL" Calcium carbonate-based Average particle size: 55µm FLAVOR : Neutral</p>	<p>Ultra-fast cleaning of difficult stains and composites with no risk of damaging them</p> <p>Note: The absence of sodium makes it possible to treat sensitive patients or those following a salt-free diet</p>
 <p>AIR-N-GO® PERIO easy Supplied with the PERIO Kit (optional)</p>		<ul style="list-style-type: none"> • Removal of the sub-gingival biofilm • Treatment of periodontal disease and peri-implantitis
 <p>AIR-N-GO® PERIO (optional)</p>	 <p>"PERIO" Glycine-based Average particle size: 25µm FLAVOR : Neutral</p>	<p>Treatment of periodontal disease and peri-implantitis</p>
 <p>AIR-N-GO® PERIO MAINTENANCE (optional)</p>		<p>Periodontal maintenance</p>

F0337/ July 2018