



# Clear Access for All Procedures

The Umbrella™ tongue, lip, and cheek retractor is the new standard in both patient comfort and stable access.



💧 Better visibility



💧 Bite check



💧 Bite registration



💧 Direct bracket bonding



💧 Dual arch impression



💧 Full arch impression



💧 Handpiece access



💧 In-office whitening



💧 Indirect bracket bonding

# Placement Guide



## STEP 1.

Simply press the tabs on the Umbrella tongue, lip, and cheek retractor together, ensuring the arrows on the top tab are pointing up, to prepare for insertion.



Do not place the retractor upside down.



## STEP 2.

Ask the patient to place the tip of their tongue on the roof of their mouth.



## STEP 3.

Choose one side of the mouth in which to start, and then comfortably insert the other side of the retractor into the cheek.



## STEP 4.

Use the tabs to center the retractor with the patient's mouth.



## STEP 5.

Check that the patient's tongue is resting comfortably behind the guard, ensuring easy access.



Do not place the tongue guard on top of the patient's tongue.

