Palodent® 360 Vollmatrizensystem

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USA: Rx only.

1. PRODUCT DESCRIPTION

The Palodent® 360 Circumferential Matrix System is designed to be used as a matrix to contain direct-placement restorative materials in posterior cavities involving proximal surfaces. The Palodent® 360 matrix band with integrated tightener/retainer allows matrix placement and retention without the need for bulky retainers or accessory placement and tightener instruments, allowing for ease of placement, better access, clear view of the operating field and greater patient comfort. Access for wedge placement (where indicated) is not hindered by an external retainer. The single-use, disposable Palodent® 360 matrix bands allow proper subgingival and proximal contact adaptation while protecting against cross-contamination.

1.1 INDICATION

The Palodent® 360 Circumferential Matrix System is indicated where placement of a matrix band around a prepared tooth will facilitate contouring of a direct placement restorative material (e.g., Class II preparations).

1.2 CONTRAINDICATIONS

None Known.

1.3 DELIVERY FORMS

The Palodent® 360 matrices are available in one thin gauge (thickness) in 3 heights: 4.5mm, 5.5mm, 6.5mm to fit a wide range of tooth sizes and shapes.

1.4 COMPOSITION

- Matrix Bands: Polished stainless steel
- Integrated tightener/retainer: Polyacetyl Thermoplastic

2. GENERAL SAFETY NOTES

Be aware of the following general safety notes and the special safety notes in other chapters of these directions for use.

Safety alert symbol
This is the safety

Technique Tips:

- To seat matrix in tight interproximal spaces – Pre-wedging or use of a Palodent® Plus or Palodent® V3 retaining ring (see Complete Directions for Use) or an Elliot Separator on posteriors may gain temporarily required space.
- To clear debris and facilitate seating – Use waxed floss or a soft wedge between tight interproximals.
- For hard to reach molars - Use locking college pliers to grip matrix along the side and facilitate matrix seating. Release pliers when matrix is securely seated on tooth.

3.5 TIGHTENING THE MATRIX

- 3.5.1 Stabilize the matrix with gentle finger pressure apically until the matrix is fully tightened. Check to be sure gingival tissue is not trapped between band and tooth surface. If so, slightly lift and reposition band prior to tightening.
- 3.5.2 Grasp tightening thumbwheel with thumb and forefinger. Rotate barrel clockwise. **NOTE:** The Palodent® 360 matrix system is designed for finger tightening ONLY. When band is snug, and resistance is felt, STOP tightening. The band is properly subgingivally tightened. Note: The integrated tightener/retainer keeps matrix locked tightly until removal is desired.

Technique Tip: Where space or grip is limited, the Palodent® 360 matrix bands may be tightened by grasping the thumbwheel and moving the retainer body forward in a ratchet motion, releasing the thumbwheel and allowing the body to return back to center, re-grasping thumbwheel and repeating until band is tight. Reverse the direction of the retainer body movement to loosen.

Technique Tip: Palodent® 360 matrix bands are compatible with Palodent® Plus or Palodent® V3 retaining rings to further stabilize and assist with proper embrasure form. When restoring an MOD preparation, two rings may be placed facing opposite directions (See Complete Directions for Use).

3.6 PLACEMENT OF RESTORATIVE

- 3.6.1 Place wedges as desired (buccal and/or lingual).
- 3.6.2 Burnish contact areas.
- 3.6.3 Place desired restorative material as per restorative material manufacturer’s directions for use.

3.7 REMOVING THE MATRIX

- 3.7.1 Following appropriate restorative curing/setting, remove retaining rings and wedges, open and remove the matrix by reversing (counterclockwise) rotation on the retainer thumbwheel until matrix is loose. This spool will usually remain attached to the band upon removal, but may become dislodged. If spool has separated from band, remove it from the mouth and properly discard.
- 3.7.2 Support the restoration marginal ridge(s) with a condenser. Slide loosened matrix buccally or lingually, while removing matrix occlusally. Properly discard matrix. *Technique Tip:* It is advisable to “post-cure” light cured restorative materials following matrix removal from all surfaces, including proximo-facial and lingual line angles.

- 

CAUTION

Small parts-ingestion/inhalation

 - Ensure matrix is grasped securely with fingers or locking college pliers during withdrawal from the mouth.
 - Recover all pieces from the mouth immediately.

- 

CAUTION

Cross-contamination

Do not reuse single use products.

Dispose of in accordance with local regulations.

Sterilization of matrices pre-use (if desired) may be accomplished (one time) as outlined below:

Limitations on processing	Cold liquid immersion disinfection/sterilization, automated washer/disinfection have not been tested or validated for efficacy and are not recommended for use.
Packaging	Paper/plastic steam sterilization pouches may be used but are not required
Sterilization	Steam autoclaving: Prevacuum: Full cycle 132°C, 4 minutes Follow manufacturer’s instructions for loading and operation cycle
Drying	Use the drying cycle of the autoclave, minimum 30 minutes.
Storage	Instruments steam autoclaved bagged should remain bagged until ready for use. Prior to use, inspect the sterilization pouch and the matrix. If the integrity of the pouch has been compromised, the matrix should be disposed of in accordance with local regulations. Instruments steam autoclaved unwrapped should be used immediately.

5. LOT NUMBER

The following should be quoted in all correspondences:

- Reorder number
- Lot number on the package
- Copy of proof of purchase

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©2018 Dentsply Sirona. Alle Rechte vorbehalten. Gedruckt in den USA.
Manufactured by Dentsply LLC 38 West Clarke Avenue Milford, DE 1996